POWER SHORTS

Produktinformation
Product information

Protection is our Profession

ORTEMA GmbH

Kurt-Lindemann-Weg 10 · 71706 Markgröningen · Germany
Phone +49 7145 - 91 53 890 · Fax +49 7145 - 91 53 981
sport-protection@ortema.de · ortema-shop.com

Version: 10/21

ORTEMA Power Shorts
ORTEMA Power Shorts

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MANUFACTURES PRODUCT INFORMATION

Technical description: Structure

The tight-fitting pants made of breathable material and the special cut, relieves the connective tissue and the muscles by compression. It allows the full range of movement in the hip area, combined with a very high wearing comfort.

Disciplines intended for: In all sports and leisure, ideal after injury of the adductors or in the groin.

Function features:

Directions for use and safety features

Due to the special cut of the materials it has been possible to achieve an excellent stabilization of the hip, groin and thighs.

Wearing

The **ORTEMA Power Shorts** are tightly fitted to the body thanks to the specific compression and offer a high wearing comfort. The **ORTEMA Power Shorts** do practically not impede any movement.

Care and maintenance

The **ORTEMA Power Shorts** are machine washable at 30 degrees. Hand wash is recommended to spare the material.

Note on safety

Hip, groin and thigh injuries can not be prevented by the pants. Careful and disciplined action greatly increases your personal safety.

Conditions of guarantee and exclusion from liability:

- damage caused by improper adjustment, misuse or wear and tear
- damage caused by incorrect repair and care
- damage caused by product modifications
- damage caused by falling, collisions or any other accidents
- damage caused by fire, water or similar factors
- parts with a limited natural life
- damage to persons and/or damage to other property
- other circumstances which are beyond the influence of the manufacturer

Größe Size	Taillenumfang Waist circumference
S	64 - 72 cm
M	72 - 80 cm
L	80 - 90 cm
XL	90 - 103 cm